# AQUATEC®

## FLEXIVIP

**SV** BRUKSANVISNING

**NO** BRUKSANVISNING

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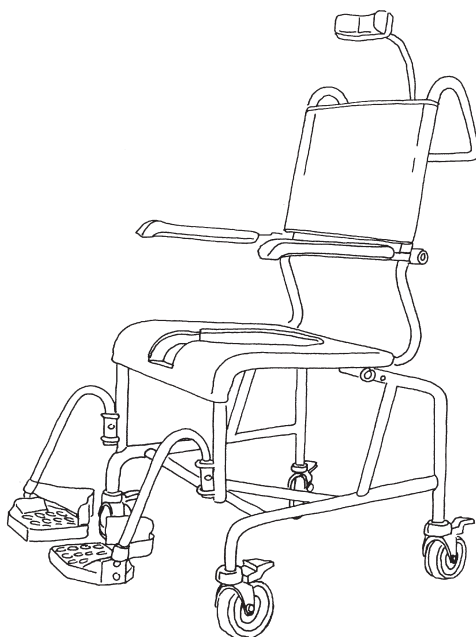
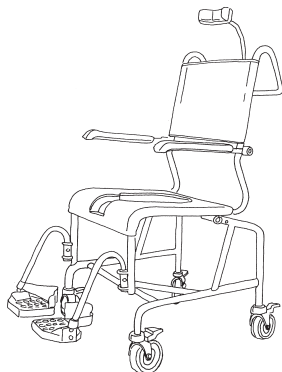
**GB** OPERATING INSTRUCTIONS

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**FR** MANUEL D' UTILISATION

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## SV BRUKSANVISNING HYGIEN-STOL

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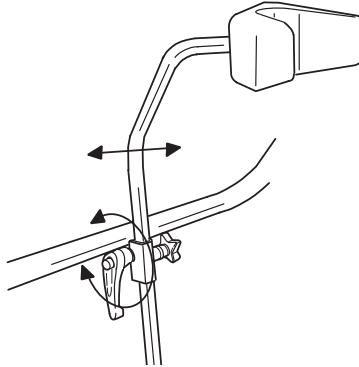
## **GB** NECK SUPPORT

### **Adjustment of angle:**

1. In order to adjust the angle of the necksupport loosen the lever. Please note that the lever is springy. Pull the lever out and turn to start and over again.
2. Adjust the neck support to required position.
3. Tighten the lever properly. Pull the lever out and twist until the lever points downwards.

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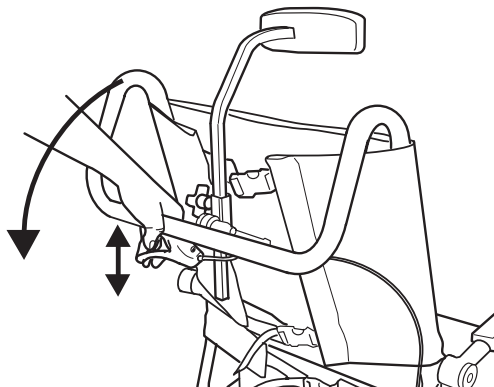
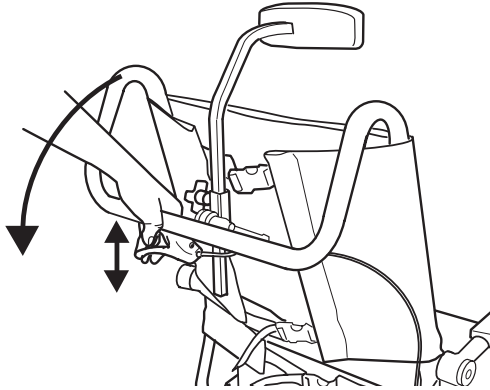
## GB TILT MECHANISM

The shower chair is to be tilted when the person is seated only, this due to the fact that the tilting is partly dependent on the persons weight. Pull the handle to tilt the chair backwards or forwards. The patient's legs must not be raised to a level exceeding the footplate level, when the shower chair is placed in a backward-tilted position. Refrain from tilting the chair in the event the chair is being used by people with little or no leg weight as this will alter the balance of the chair. Tilting in these cases can only be made if the chair is equipped with tilt-stoppers (accessory).

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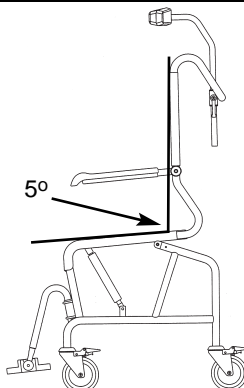
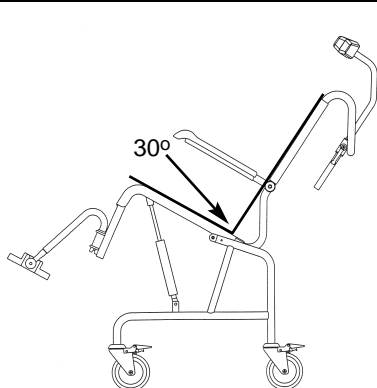
**AQUATEC FLEXIVIP** can be tilted

30° backwards.

**AQUATEC FLEXIVIP** can be tilted  
5° forwards.

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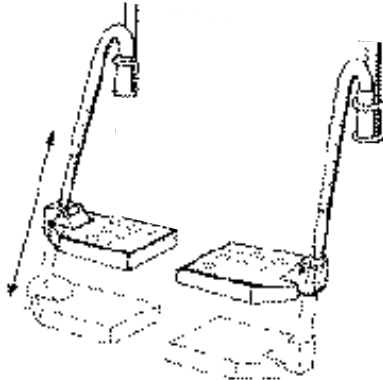
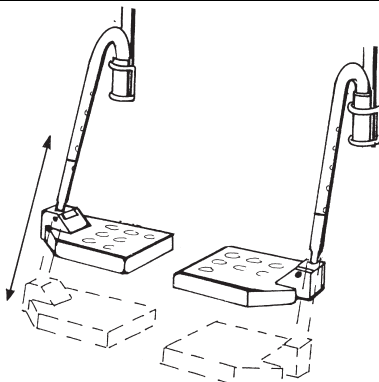
## GB FOOT RESTS

1. To adjust the foot rest press the button inside the vertical tube of the foot rest.
2. Adjust the foot rest to required position. To secure that the foot rests are fixed this is marked by a clear sound and the button will pop out..
3. Lift up the foot rest and swing to the side.
4. Pull upwards to remove the foot rests.
5. The footplate can be angled or folded up.

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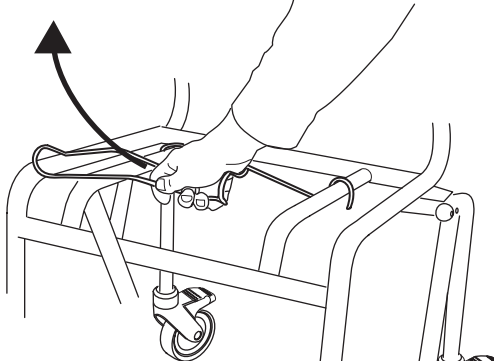
### **GB** HOLDER FOR SANITARY PAN

The holder for the sanitary pan is removable. Pull the seat upwards and lift off the holder.

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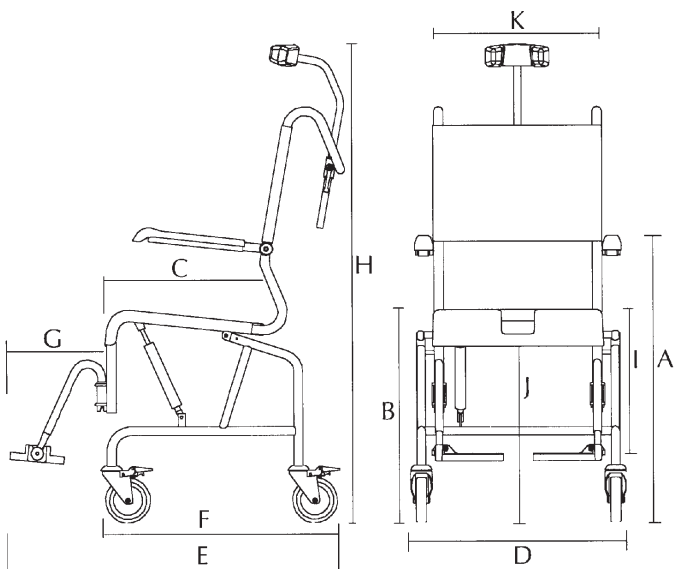
**GB****INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKING:**

When the product can no longer be used, it and its accessories as well as the packaging material should be assorted in the correct way. The individual parts can be separated and handled according to the list specification.

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## GB MAINTENANCE INSTRUCTIONS

**User's maximum weight 275 lbs / 125 kg.**

**AQUATEC FLEXIVIP** tiltchair is intended for use as a shower chair and/or a bedside toilet commode. It is not for use in pool or a similar environment where equipment is immersed in water.

Minimum two wheels should be locked in position while toileting or showering. **FLEXIVIP** Mobile Shower chair is only for use on flat surfaces. For optimum stability, position the link wheels forwards as shown in the fig. see page 13.

While transporting person in **AQUATEC FLEXIVIP**, the seat must be placed in horizontal position. Refrain from tilting the chair in the event the chair is being used by people with little or no leg weight as this will alter the balance of the chair. Tilting in these cases can only be made if the chair is equipped with tilt-stoppers (accessory).

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the hygiene seat with solvent-free household cleaner. Keep the wheels clean.
- **Ensure that screws and adjustments** are properly tightened.
- Lubricate the wheels regularly with grease.
- **Immediately contact the nearest dealer if a defect should appear. Never use a defective chair.** Do not attempt repairs yourself!

**DOLOMITE** will **not** accept responsibility for adaptations, faulty repairs, etc. that are outside our control. Refrain from tilting the chair in the event the chair is being used by people with little or no leg weight as this will alter the balance of the chair. Tilting in these cases can only be made if the chair is equipped with tilt-stoppers (accessory).

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## **DOLomite AB**
**AQUATEC GmbH & Co.KG**
**MOBITEC Mobilitätshilfen GmbH**